© 2009 Ultradent Products, Inc. All rights reserved.
 U.S. Patent Numbers: 5,376,006; 5,409,631; 5,575,654;
 5,746,598; 5,770,105; 5,851,512; 5,863,202; 5,980,249;
 5,985,249; 6,306,370; 6,306,379; 6,309,625; 6,368,576;
 6,312,671; 6,322,774; 6,500,408; 6,730,316; 6,860,736;
 6,964,571; 7,004,756; 7,040,897; 7,128,899; 7,192,280 and
 7,247,022. Australian Patent Number: 654955 and 780274.
 Canada Patent Number 2078776;
 Korean Patent Number: 10-0734581;
 Russian Federation 008520;
 South Africa Patent Number: 2005/09141.
 New Zealand Patent Number: 543633.
 Ukraine Patent Number: 82878.
 Other U.S. and International patents granted and pending.
 For Professional use only.
 For an MSDS of this product, please see: www.ultradent.com

	°F	°C	EN-Refrigerate
			DE-Kühlen
			FR-Réfrigérer
			NL-Koelen
			IT-Refrigerare
			ES-Refrigerar
			PL-Przechowywać w temperaturze pokojowej
			PT-Refrigerar
			SV-Förvara i kylskåp
			DA-Opbevares køligt
			FI-Säilytä kylmassä
			EL-Να φυλάσσεται στο ψυγείο

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'U' • Upper bleaching tray
'L' • Lower bleaching tray
Product must be refrigerated. Before using, allow product to warm to room temperature. This will make it easier for the tray to adhere to the teeth. Remove from package.

Center tray on arch.

Gently suck down on the tray.

Remove outer tray.

Lightly tap tray if necessary.

Wear for 30-60 minutes per day. When finished remove bleaching tray and brush teeth.

Precautions:

1. Keep tooth bleaching product away from small children.
2. A dental examination is recommended prior to bleaching.
Some discolorations may indicate the need for dental treatment other than bleaching. Porcelain crowns, veneers, and white fillings do not lighten upon bleaching.
3. Only intended for use by those with healthy gum tissue. Gum tissue irritation may result if used in individuals with periodontal disease
4. This product is NOT for nighttime use.
5. Occasionally, some parts of teeth may bleach faster or less evenly than others. Results may be varied. In approximately 5% of the population, teeth may be resistant to bleaching. **BLEACHING TEETH WITH LARGE METAL FILLINGS MAY CAUSE THE METAL COLOR TO SHOW THROUGH THE TEETH.**
6. Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
7. Do not use tobacco products or eat while bleaching.
8. Not intended for use by pregnant women.
9. Keep gel out of heat/sunlight. Do not freeze.
10. Do not swallow gel or rinses. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.

WARNING: Some users may experience sensitivity, which if it occurs is most often mild and transient, but in some cases may require treatment by your dentist. If you experience significant discomfort, stop using this product and consult your dentist. If you have any questions about this product or treatment, please ask your dentist.

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